








# DURADENT SYSTEMS


## Instructions for the TubeTray® by DuraDent

<p>1. Press a TubeTray top and base together. This is to prevent any excess material from getting into critical areas. Pour die stone into TubeTray reservoir until level with the top lip.</p>	
<p>2. Pour die stone into the dental impression until filled.</p>	
<p>3. Turn impression over and carefully place on filled TubeTray reservoir.</p>	
<p>4. When the die stone has completely set, remove the impression. Remove the top section from the base by inserting a DuraDent screwdriver into the front end and gently pressing in a downward motion.</p>	
<p>5. Section and trim the dies as needed.</p>	
<p>6. Place the sectioned dies on the TubeTray base. Easy, neat, no pins or accessories!</p>	
<p>7. Proceed with articulation. The TubeTray can be used with virtually any disposable or reusable articulator.</p>	

## Helpful hints

- Using a sharp blade, trim away any excess material from the impression before pouring.
- Wipe away any excess die stone from the TubeTray reservoir and impression while the die stone is setting. This minimizes the need for grinding the die stone after it has set.
- Use a DuraDent Rubber Cutting Block to make sectioning easier.
- For best results, while sectioning, let the saw do the work - don't press.

- After sectioning, brush away any excess material with a Nylon Cleanup Brush

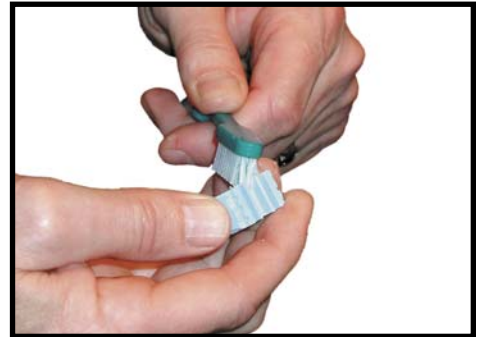

- To easily lift a die section from the base, insert the DuraDent Screwdriver into one of the side slots and gently twist the screwdriver

